

CROSSVISION PLUS

"CROSSVISION PLUS" is a physiological pH sterile solution for ophthalmic use containing cross-linked hyaluronic acid and distilled waters of chamomile and mallow.

Cross-linked hyaluronic acid, made using patented technology, preserves all the properties of linear hyaluronic acid while having structural characteristics that give greater stability and longer corneo-conjunctival permanency than its predecessor, as it is degraded more slowly by hyaluronidase enzyme.

The cross-linked structure of hyaluronic acid ensures better hydration and stabilises the tear film. It is therefore recommended for treating dry eyes.

"CROSSVISION PLUS" also contains: distilled waters of Chamomile and Mallow to ensure moisturising action, sodium Chloride and phosphate buffer to ensure a physiological solution.

This formula has been developed to give maximum comfort from the first application. It is recommended for everyday use, day or night.

"CROSSVISION PLUS" is compatible with soft contact lenses.

INTENDED USE

"CROSSVISION PLUS" is recommended for:

- Restoring and improving tear film stability in case of dry eye disease.
- Soothing eye redness, dryness and discomfort caused by:
 - Environmental factors (air conditioning, excessive exposure to sunlight, wind, smoke, pollution, sea or swimming pool water and sea air);
 - Eye strain caused by: studying, prolonged computer use;
 - Abnormal lacrimation following inflammation;
 - Abnormal lacrimation following eye surgery;
 - Use of soft or semi-rigid contact lenses;
 - Refractive surgery.

COMPOSITION

Cross-linked Sodium Hyaluronate aqueous solution (of which 0.3% cross-linked sodium hyaluronate), distilled water of Chamomile, distilled water of Mallow, sodium phosphate dibasic dodecahydrate, sodium phosphate monobasic monohydrate, sodium chloride, water for injections. Contains no preservatives.

INSTRUCTION FOR USE

- Wash and dry your hands thoroughly before use;
- Remove the safety seal before use;

- Remove the cap to open the bottle;
- Upright the bottle and gently squeeze 2 drops into each eye;
- Wait a few seconds after each drop without squeezing the bottle. This will enable enough air to flow back into the sterile air filter system, otherwise it is more difficult to get the drops out;
- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions. A small quantity of solution also always remains in the bottle;
- Shake off the remaining drop from the tip after use;
- After use close immediately the bottle with the cap;
- Do not put the protection cap back on when the container is still squeezed;
- Do not touch the applicator tip with anything, including your eye or your fingers;
- After first use, the bottle contents should be used within 3 months.

WARNINGS AND PRECAUTIONS

- If you suffer from severe damage to the clear layer at the front of the eye (the cornea), phosphates may cause in very rare cases cloudy patches on the cornea due to calcium build-up during treatment;
- Strictly personal use only;
- Ophthalmic use only;
- Do not use the product if the container is damaged or unsealed when first purchased;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch the eye or any other surface with the tip of the applicator;
- Do not use in case of known incompatibility towards any of the components of the solution and/or Asteraceae plants and/or Malvacee plants;
- The product is not suitable for children;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- If side effects occur during use, stop using the product and consult a doctor;
- Do not use the product after the expiry date shown on the pack;
- Dispose of the product according to your local waste management regulations;
- Keep out of reach of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- Do not try to tamper or disassemble the nozzle from the bottle;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

SIDE EFFECTS

Immediately after application, the vision may be slightly impaired for a short time. It is therefore advisable to wait for a few seconds before any activity that requires clear vision.

STORAGE CONDITIONS

Keep the product dry between 2°C and 30°C and away from direct heat and sunlight. The contents may be reused within 3 months of first opening.

PACKAGING

The package includes a multidose bottle with OSD (ophthalmic squeeze dispenser) cap, containing 10 mL of ophthalmic solution.

VALIDITY

If unopened, do not use the product after the expiry date shown on the pack and on the label.



DT/01/DT/004



C.O.C. Farmaceutici S.r.l.

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Last Revision: 02.01 del 10/2022



C.O.C. Farmaceutici srl
Via Modena, 15
40019 - S. Agata Bolognese (BO) ITALIA
Via Chiesa Sud, 156 C/D/E/F/G
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1200.000 I.V.
C.F. e Iscr. Reg.
Impr. di Bologna 03541870378
PIVA IT 01603491208
R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.
Via Costarica, 20/22
00071 Pomezia (Roma)
ITALIA

cap. soc. € 472.500,00 I.V.
C.F. / P.IVA 05406661008
R.E.A. Roma N. 883476