

# HYALUMILLA

"HYALUMILLA" is a sterile solution for ophthalmic use with physiological pH, dispensed in single-dose containers of 0.5 ml, preservative free.

The main action of the device is to lubricate and moisturize the ocular surface, thanks to sodium hyaluronate. "HYALUMILLA" is indicated in case of dry or irritated eyes.

Sodium hyaluronate contained in the solution provides to the product mucomimetic and viscoelastic properties such as to ensure adequate lubrication of the ocular surface, protecting it from friction caused by eye movements. For this reason, it soothes irritated and dry eyes. Furthermore, the polymer is able to retain water and simultaneously interact with the ocular epithelium. The solution is therefore able to maintain the ocular surface humidified, refreshes dry and irritated eyes and increases the stability of precorneal tear film that protects the surface from environmental agents.

The formula also contains: distilled waters of Ginkgo biloba, Indian Penny Wort, Blueberry and Fennel to ensure moisturizing action, Sodium Chloride and two buffers agents (boric acid and sodium tetraborate decahydrate) to ensure a physiological solution.

"HYALUMILLA" can be used by contact lenses wearers. It is compatible with contact lenses.

## INTENDED USE

Repairing and protective ocular drops indicated to refreshing, soothing and lubricating action for dry eyes or in cases of irritation of the eye or the ocular mucosa due to:

- Environmental factors (air conditioned, excessive exposure to solar radiation, wind, smoke, pollution).
- Dryness of the eyes due to excessive fatigue caused by studying or working on the computer for lengthy periods, or dry eye syndrome.
- Ophthalmic stress following surface eye surgery.
- Wearing of contact lenses many hours a day.

## COMPOSITION

Sodium hyaluronate 0,4%, Ginkgo biloba (*Ginkgo biloba*) distilled water, Blueberry (*Vaccinium myrtillus*) distilled water, Fennel (*Foeniculum vulgare*) distilled water, Indian Penny Wort (*Centella Asiatica*) distilled water, Boric acid, Sodium tetraborate decahydrate, Sodium chloride, water for injections.

## INSTRUCTION FOR USE

- Wash and dry your hands thoroughly before use;
- Open the protective aluminum pouch;
- Detach the single dose container from the strip;
- Twist and remove the cap to break the seal;
- Place the cap taking care to put it on a clean surface;
- Instill 1/2 drops in each eye by gently squeezing the single-dose container; no more than 10 doses (1-2 drops per eye) should be applied per day;
- After use immediately close the single-dose container: content must be used within 12 hours.

## WARNINGS AND PRECAUTIONS

- The product contains the following CMR substances in concentrations above 0.1% w/w: boric acid and sodium tetraborate decahydrate. The product is not suitable for children, pregnant and breastfeeding women;
- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if container is damaged or not perfectly closed when you open it for the first time;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- Do not use the product after the expiry date indicated on the package;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use in case of known incompatibility towards any of the components of the solution and/or Ericaceae plants and/or Apiaceae plants and/or Anacardiaceae plants and/or Poison Ivy and/or celery;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- Dispose of the product according to your local waste management regulations;
- Keep out of the reach and sight of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

## SIDE EFFECTS

No undesirable side-effects are known so far.

## STORAGE CONDITIONS

Store below 25°C.

Content must be used within 12 hours after the first opening.

Store unused single doses in the aluminum pouch.

## PACKAGING

From 5 to 30 re-closable single dose of 0.5 ml

## VALIDITY

See the expiry date reported on the box and on the single doses.



DT/01/DT/003



**C.O.C. Farmaceutici S.r.l.**

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Last Revision: 02.01 del 10/2022



**C.O.C. Farmaceutici srl**  
Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**  
Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476