

# CROSSVISION LIGHT

Based upon cross-linked hyaluronic acid

10 mL MULTI DOSE PRESERVATIVE FREE

**Crossvision light** is a sterile solution for ophthalmic use with a physiological pH, based upon cross-linked hyaluronic acid. Hyaluronic acid is a mucopolysaccharide which is evenly distributed on the ocular surface after blinking thanks to its pseudoplastic and mucomimetic properties, while the cross-linking agent acts synergistically enhancing and prolonging the humectant action. **Crossvision light** hereby obtains a high level of hydration and persistence on the eye surface, able also to promote a higher tear film stability and a better and more rapid repair and regeneration of the corneal epithelium. **Crossvision light** is therefore used for its lubricating, prolonged moisturizing and reparative actions as eye drops able to give relief to dry, sore or tired eyes.

**Crossvision light** relieves redness, fatigue and discomfort to the eyes and ocular mucosa caused by environmental factors such as wind, smoke, pollution, prolonged exposure to sunlight, dried salt, sea water or pool water, or due to intense commitment to study, work, prolonged use of the computer or for a long period of driving. Hydrates and lubricates the cornea and the conjunctiva in case of dry eye due to the use of contact lenses.

**Crossvision light** gives a high degree of prolonged hydration over time and protects the corneal surface. It's therefore indicated as an adjuvant in dry eye syndrome and especially after refractive surgery. The formula was researched to ensure maximum comfort already in the first moments after the application. Suitable for everyday use and evening.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

**Crossvision light** is suitable with contact lenses.

## INTENDED USE

**Crossvision light** is indicated to give relief to the eyes in cases of reddening, fatigue and discomfort of the eye or the ocular mucosa due to:

- environmental factors (air conditioned, artificial heating, excessive exposure to solar radiation, wind, smoke, pollution, sea water or pool water and dried salt);
- dryness of the eyes due to excessive fatigue caused by working on the computer for lengthy periods, study or dry eye syndrome;
- wearing soft or semirigid contact lenses.

In addition, after doctor's advice, it can be used in the following cases:

- lacrimal abnormalities secondary to chronic blepharitis and quality or quantity tears alterations known as "dry eye syndrome";
- lacrimal abnormalities secondary to eye surgery of various nature or eyelid aesthetic surgery;

- improvement lacrimal function stability after refractive surgery.

## COMPOSITION

Sodium hyaluronate crosspolymer aqueous solution (of which sodium hyaluronate crosspolymer 0,02%), Sodium phosphate dibasic dodecahydrate, Sodium phosphate monobasic monohydrate, Sodium chloride, W.F.I.. Preservative free.

## INSTRUCTION FOR USE

- Cleanse your hands thoroughly;
- tear off the tamper-evident seal before the first administration;
- remove the protective cap and open the bottle;
- turn the bottle upside down and exert slight pressure on the bottle to instill 2 drops in each eye, as required or according to the advice of your doctor;
- the gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the proprieties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions;
- after use shake off the remaining drop and immediately re-close the bottle;
- the content should be re-used within the next 3 months from the first opening.

## WARNING AND CAUTION

- The product is for ophthalmic use only.
- The product is intended for single-patient use.
- Use the product before applying your contact lenses.
- Do not use the product if the container is damaged or not perfectly closed when you open it for the first time.
- During the instillation, do not touch the eye or any other surface with the dropper of the container.
- Do not use the product if you are allergic or in case of known hypersensitivity to any of the ingredients.
- In case of problems during treatment interrupt the use and consult a doctor.
- Dispose of properly the container after use.
- Do not use the product after the expiry date indicated on the package and on the label.
- Keep away from children.
- The administration of ophthalmic drugs concurrently with the eye drops should be carried out under the supervision of your doctor who can evaluate the possible interactions and should be carried out at least 15 minutes after the ocular drops are instilled.
- Temporary slight blurring of vision may occur immediately after use of product.
- In case of eye irritation during treatment interrupt the use and consult a doctor.
- Do not swallow.
- The product is usually intended for continuous use for no more than 30 days; consult your doctor if symptoms persist.
- Consult your doctor before you use this product if you are pregnant or breast-feeding.

### Side effects

No side effects were reported after use of the product.

## PACKAGE

The package is composed of a bottle with polidose OSD locking system (ophthalmic squeeze dispenser), containing 10 ml of product free of preservatives.

## STORAGE CONDITIONS

Store between 2°C and 30°C in a dry place away from heat and direct exposure to sunlight. The content must be used within 3 months after the first opening.

## VALIDITY

With un-open pack, not exceed the expiry date stated on the box and on the label.

**C.O.C. Farmaceutici S.r.l.**  
Via Modena, 15 - 40019  
Sant'Agata Bolognese (Bo)  
Italy

 **0546**  
MEDICAL DEVICE

Distributed by: \_\_\_\_\_  
\_\_\_\_\_

**STERILE A**

**Last Revision: 05.01 on 11/2020**



**C.O.C. Farmaceutici srl**  
Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**  
Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476