

# HYDRAMILLA HY 0.3%

**“based upon hydroxypropylmethylcellulose”**

From 5 to 20 re-closable single-dose of 0.5 ml

**Hydramilla HY 0,3%** is a sterile, ophthalmic soothing and emollient solution, with physiological pH, and based upon hydroxypropylmethylcellulose.

Hydroxypropylmethylcellulose is a viscous polymer with wetting and muco-mimetic properties which can stabilize the tear film on the surface of the eyes by creating a protective, transparent and viscoelastic shield.

Thanks to its proprieties hypromellose protects and lubricates the ocular surface reducing discomfort sensation and providing soothing comfort to irritated eyes by re-establishing natural tears with a long-lasting relief.

**Hydramilla HY 0,3%** decreases symptoms of dry eyes such as burning, itching, and foreign body sensation.

**Hydramilla HY 0,3%** acts as humectant and lubricant, providing relief of dry eye conditions, including those associated with the use of PC, infrequent blinking, atmospheric pollution, air conditioning, central heating, wind and sun.

**Hydramilla HY 0,3%** also ensures relief from ophthalmic stress due to the use of contact lenses.

**Hydramilla HY 0,3%** is compatible with contact lenses and does not contain preservative.

**Hydramilla with 0,3% of hypromellose** can be used in case of a moderate dry eye syndrome or dry eyes symptoms or more commonly, in case of discomfort sensation.

## INSTRUCTION FOR USE

- Cleanse your hands thoroughly.
- Open the aluminum pouch.
- Detach the single-dose container from the strip and open the single-dose container by turning and then lifting off the closing cap.
- Instill 1/2 drops in each eye by gently squeezing the single dose.
- After use immediately close the single-dose container: the content must be used within 12 hours after the first opening.
- Also indicated for frequent use, even several times a day (3 or 4 time or more often if required) or according the doctor indication.

Vision may be temporarily blurred after use of product. Also, minor burning/stinging/irritation may temporarily occur as a consequent of the ocular irritation. If any of these effects persist or worsen, tell your doctor promptly.

## COMPOSITION

Hydroxypropyl methylcellulose (HPMC), sodium phosphate monobasic monohydrate, sodium phosphate dibasic dodecahydrate, sodium chloride, W.F.I.. Preservative free.

## WARNING AND CAUTION

- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Use the product before applying your contact lenses.
- Never use the product if the aluminium pouch or the container results, at the time of first opening, damaged or not perfectly sealed.
- During the application, do not touch your eyes and any other surface with the dropper of the container.
- Do not use in the event of known incompatibility with any of the components.
- The administration of ophthalmic drugs concurrently with eye drops should be carried out under the supervision of the doctor who can evaluate the possible interactions and should be carried out at least 15 minutes after the ocular drops are instilled.
- In case of problems during treatment interrupt the use and consult a doctor.
- Do not use after the expiry date indicated on the package.
- Dispose of properly the container after use.
- Keep away from children.
- Not to swallow.
- The product is usually intended for continuous use for no more than 30 days; consult your doctor if symptoms persist.
- Consult your doctor before you use this product if you are pregnant or breast-feeding.
- **Hydramilla HY 0,3%** is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention.

## PACKAGE

From 5 to 20 re-closable single-dose of 0.5 ml

## STORAGE CONDITIONS

Do not store above 30°C.

The content must be used within 12 hours after the first opening.

Store the unused single dose into the original aluminium pouch

## VALIDITY

With un-open pack: not exceed the expiry date stated on the box and on the single dose.

**C.O.C. Farmaceutici S.r.l.**  
Via Modena, 15 - 40019  
Sant'Agata Bolognese (Bo)  
Italy

**CE** 0546  
MEDICAL DEVICE

Distributed by: \_\_\_\_\_  
\_\_\_\_\_

**STERILE A**

**Last Revision: 07.01 on 01/2021**



**C.O.C. Farmaceutici srl**  
Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**  
Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476