

# HYDRAMILLA HY 0.5%

## Ocular drops based upon hypromellose

10 ml Multi dose preservative free

**Hydramalla HY 0.5%** is a sterile, ophthalmic soothing and emollient solution, with physiological pH, and based upon hypromellose.

Thanks to the OSD closure system - able to preserve the microbiological, chemical and physical characteristics of the solution - **Hydramalla HY 0.5%** does not contain preservative.

Hypromellose is a viscous polymer with wetting and muco-mimetic properties which can stabilize the tear film on the surface of the eyes by creating a protective, transparent and viscoelastic shield.

Thanks to its properties, hypromellose protects and lubricates the ocular surface reducing discomfort sensation and providing soothing comfort to irritated eyes by re-establishing natural tears with a long-lasting relief.

**Hydramalla HY 0.5%** decreases symptoms of dry eyes such as burning, itching, and foreign body sensation.

**Hydramalla HY 0.5%** acts as humectant and lubricant, providing relief of dry eye conditions, including those associated with the use of PC, infrequent blinking, atmospheric pollution, air conditioning, central heating, wind and sun.

**Hydramalla HY 0.5%** also ensures relief from ophthalmic stress due to the use of contact lenses.

**Hydramalla HY 0.5%** is compatible with contact lenses.

**Hydramalla with 0.5 % of hypromellose** can be suggested, under medical supervision, in case of a severe dry eye syndrome, as an adjunct to disease-modifying therapies.

## INSTRUCTION FOR USE

- tear off the tamper-evident seal before the first administration;
- remove the protective cap and open the bottle;
- instill 1/2 drops in each eye slightly pressing the bottle;
- the gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the proprieties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions;
- after use, shake the bottle once downwards in order to remove any excess before replacing the cap on the container and close the bottle;
- it is suitable for frequent use, even several times a day (3 or 4 time or more often if required) or according the doctor indication.

Vision may be temporarily blurred after use of the product. Also, minor burning/stinging/irritation may temporarily occur as a consequent of the ocular irritation. If any of these effects persist or worsen, tell your doctor promptly.

## COMPOSITION

Hydroxypropylmethylcellulose (HPMC), sodium phosphate monobasic monohydrate, sodium phosphate dibasic dodecahydrate, sodium chloride, W.F.I.

## WARNING AND CAUTION

- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Use the product before applying your contact lenses.
- Never use the product if the container results, at the time of first opening, damaged or not perfectly sealed.
- During the application, do not touch your eyes and any other surface with the dropper of the container.
- Do not use in the event of known incompatibility with any of the components.
- The administration of ophthalmic drugs concurrently with the eye drops should be carried out under the supervision of your doctor who can evaluate the possible interactions and should be carried out at least 15 minutes after the ocular drops are instilled.
- In case of problems during treatment interrupt the use and consult a doctor.
- Do not use after the expiry date indicated on the package and label.
- Dispose of properly the container after use.
- Keep away from children.
- Do not swallow.
- The product is usually intended for continuous use for no more than 30 days; consult your doctor if symptoms persist.
- Consult your doctor before you use this product if you are pregnant or breast-feeding.
- **Hydramilla HY 0.5%** is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention.

## PACKAGE

10 ml Multi dose bottle, preservative free.

## STORAGE CONDITIONS

Do not store above 30°C.

After the first opening the content must be used within 3 months.

## VALIDITY

With un-open pack: not exceed the expiry date stated on the box and on the multi dose bottle.

**C.O.C. Farmaceutici S.r.l.**  
Via Modena, 15 - 40019  
Sant'Agata Bolognese (Bo)  
Italy

**CE** 0546  
MEDICAL DEVICE

Distributed by: \_\_\_\_\_  
\_\_\_\_\_

**STERILE A**

**Last Revision: 05.01 on 01/2021**



**C.O.C. Farmaceutici srl**  
Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**  
Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476