

MIRTHILLA SINGLE DOSE

From 5 to 20 re-closable single-dose of 0.5 ml
Compatible with contact lenses

READ CAREFULLY BEFORE START TO USE THE PRODUCT

Mirthilla is a sterile, ophthalmic lasting moisturizing and lubricating solution, with physiological pH, based upon hypromellose and sodium hyaluronate. The absence of preservatives allows the use of the solution by contact lens wearers. Administer the product before wearing contact lenses.

Hypromellose is a viscous polymer with wetting and muco-mimetic properties which can stabilize and re-establish the tear film on the surface of the eyes by creating a protective, transparent and viscoelastic shield. Hypromellose, thanks to its properties, protects and lubricates the ocular surface reducing discomfort sensation and providing comfort to irritated eyes.

Also sodium hyaluronate is a viscous polymer and it is used for its moisturizing and lubricating effect, which counteracts dry eyes and creates, as the hypromellose, a film that protects the ocular mucosa from external agents and from damage by free radicals.

Mirthilla relieves symptoms of dry eyes such as redness, burning, itching, and foreign body sensation, helping the restore of the natural ocular film.

Mirthilla provides a lasting moisture and lubricant effect on the ocular surface. The solution relieves dry eye conditions, including those associated with the use of PC, infrequent blinking, certain medical treatments, atmospheric pollution, dry atmospheres (e.g. air conditioning, wind and sun) and ophthalmic stress due to the use of eye lenses.

INSTRUCTION FOR USE

- Cleanse your hands thoroughly.
- Open the aluminum pouch.
- Detach the single-dose container from the strip and open the single-dose container by turning and then lifting off the closing cap.
- Instill 1/2 drops in each eye by gently squeezing the single dose.
- After use immediately close the single-dose container: the content must be used within 12 hours after the first opening.
- It is suitable for frequent use. Unless your doctor has told you differently, you can use the product even several times a day (2 or 3 times or more often if required) to provide sufficient hydration. As dry eye is a condition which varies from person to person, the drops should be used as often as you think necessary, to relieve your dry eye.

COMPOSITION

Sodium hyaluronate, hydroxypropyl methylcellulose, boric acid, sodium tetraborate decahydrate, sodium chloride, W.F.I. Contains no preservatives or phosphates.

WARNINGS AND CAUTIONS

- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Never use the product if the container results, at the time of first opening, damaged or not perfectly sealed.
- During the application, do not touch your eyes and any other surface with the dropper of the container.
- Contraindicated in patients with a history of hypersensitivity to any of the components.
- In case of problems during treatment interrupt the use and consult a doctor.
- Do not use after the expiry date indicated on the package and label.
- Dispose of the container properly after use.
- Keep away from children.
- Leave at least 15 minutes between administrations of any other ophthalmic drugs.
- Remove contact lenses before using this product.
- Pregnant, breast-feeding women and those who have recently undergone ophthalmic surgery must consult their doctor before use.

COMMON SIDE EFFECT

- The product can cause a short term blurring of vision after use. Do not drive or use machines until your sight is clear;
- Occasionally these eye drops may cause: redness, tingling, foreign body sensation and burning in the eyes, stickiness of the eyelashes, photophobia, tearing;
- Prolonged use can give rise to sensitization phenomena.

If the effects persist or worsen, contact your doctor promptly.

PACKAGE

From 5 to 20 re-closable single-doses of 0.5 ml.

STORAGE CONDITIONS

Do not store above 30°C.

The content must be used within 12 hours after the first opening.

Store the unused single dose into the original aluminum pouch.

VALIDITY

With un-open pack, not exceed the expiry date stated on the box and on the single dose.



C.O.C. Farmaceutici S.r.l.

Via Modena, 15
40019 - Sant'Agata Bolognese (BO)
Italy



MEDICAL DEVICE

STERILE A

DISTRIBUTED BY:

Last Revision 03.01 of 03/2022



C.O.C. Farmaceutici srl

Via Modena, 15
40019 - S. Agata Bolognese (BO) ITALIA
Via Chiesa Sud, 156 C/D/E/F/G
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1200.000 I.V.
C.F. e Iscr. Reg.
Impr. di Bologna 03541870378
P.IVA IT 01603491208
R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.

Via Costarica, 20/22
00071 Pomezia (Roma)
ITALIA

cap. soc. € 472.500,00 I.V.
C.F. / P.IVA 05406661008
R.E.A. Roma N. 883476