

HYALUMILLA ADVANCED

"HYALUMILLA ADVANCED" is a sterile solution for ophthalmic use with a physiological pH, based upon pharmaceutical grade Hyaluronic Acid (as sodium hyaluronate), a polymer normally present in different areas of the human body. Hyaluronic acid has viscoelastic, muco-mimetic and muco-adhesive properties that improve tear film stability and maintain ocular surface lubricated and hydrated.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions. "HYALUMILLA ADVANCED" is compatible with soft and rigid contact lenses.

INTENDED USE

"HYALUMILLA ADVANCED" moisturizes and lubricates eye relieving symptoms of irritation, dryness and foreign body sensation caused by:

- - dry eye syndrome;
- - mechanical factors (contact lenses use, after eye surgery);
- - environmental factors (wind, smoke, pollen, dust, pollution);
- - visual stress (excessive light, prolonged use of computers, pc and tablets).

COMPOSITION

Sodium Hyaluronate 0,2%, Citric Acid Monohydrate, Trisodium Citrate Bihydrate, Sodium chloride, Water For Injection.

INSTRUCTION FOR USE

- Wash and dry your hands thoroughly before use;
- Tear off the tamper-evident seal before the first administration;
- Remove the protective cap;
- Tilt your head back and look up;
- Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye;
- Take the bottle with other hand, turn the bottle upside down and apply moderate pressure to the bottle to release 1 drop of "HYALUMILLA ADVANCED" into the pocket;
- Wait a few seconds after each drop without squeezing the bottle. This will enable enough air to flow back into the sterile air filter system, otherwise it is more difficult to get the drops out;

- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions. A small quantity of solution also always remains in the bottle;
- If you think that you have missed the eye, then insert another drop;
- Repeat the process for the other eye, if required;
- Shake off the remaining drop from the tip after use;
- After use close immediately the bottle with the cap;
- Do not put the protection cap back on when the container is still squeezed;
- Do not touch the applicator tip with anything, including your eye or your fingers;
- "HYALUMILLA ADVANCED" can be used as required or according doctor's advice;
- "HYALUMILLA ADVANCED" can be used up to 3 months after first opening.

WARNINGS AND PRECAUTIONS

- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if container is damaged or not perfectly closed when you open it for the first time;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- Do not use the product after the expiry date indicated on the package;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use in case of known incompatibility towards any of the ingredients;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- The product is usually intended for continuous use for no more than 30 days; if symptoms persist consult your doctor.
- Dispose of the product according to your local waste management regulations;
- Keep out of the reach and sight of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- Do not try to tamper or disassemble the nozzle from the bottle;
- "HYALUMILLA ADVANCED" is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

SIDE EFFECTS

Immediately after instillation, the product may cause a slight temporary blurring of vision due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before any activity that requires clear vision.

STORAGE CONDITIONS

Store between 2°C and 30°C.

Content must be used within 3 months after the first opening.

PACKAGING

One bottle containing 10 ml of preservative free sterile solution.

VALIDITY

See the expiry date reported on the box and on the label.



DT/01/DT/006



C.O.C. Farmaceutici S.r.l.

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Last Revision: 02.01 del 10/2022



C.O.C. Farmaceutici srl

Via Modena, 15
40019 - S. Agata Bolognese (BO) ITALIA
Via Chiesa Sud, 156 C/D/E/F/G
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.
C.F. e Iscr. Reg.
Impr. di Bologna 03541870378
P.IVA IT 01603491208
R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.

Via Costarica, 20/22
00071 Pomezia (Roma)
ITALIA

cap. soc. € 472.500,00 I.V.
C.F. / P.IVA 05406661008
R.E.A. Roma N. 883476