

# BLUEMILLA

"BLUEMILLA" is a sterile, ophthalmic moisturizer and lubricant solution, with physiological pH, based on sodium hyaluronate and dispensed in a multi dose bottle.

Sodium hyaluronate is used for its moisturizing and lubricating effect, which counteracts dry eyes and creates a film that protects the corneal epithelium from environmental agents. The formula contains also Sodium Chloride and phosphates buffers, sodium phosphate dibasic heptahydrate and sodium phosphate monobasic dihydrate to guarantee a physiological solution and distilled waters of Chamomile and Cornflower to ensure moisturizing action.

## INTENDED USE

"BLUEMILLA" is indicated for dry eyes or discomfort of the eye or ocular mucosa due to:

- environmental factors (air conditioning, excessive solar radiation, wind, smoke, pollution, sea water or swimming pool water);
- dry eye resulting from excessive fatigue due to long hours working on computers, studying or due to dry eye syndrome;
- use for many hours a day of contact lenses.

## COMPOSITION

Sodium hyaluronate 0.1%, distilled water of cornflower, distilled water of chamomile, sodium phosphate bibasic hepta-hydrate, sodium phosphate monobasic di-hydrate, sodium chloride, water for injections. Preservative-free.

## INSTRUCTION FOR USE

- Wash and dry your hands thoroughly before use;
- Tear off the tamper-evident seal before the first administration;
- Remove the protective cap;
- Press the container and instill 2 or 3 drops of "BLUEMILLA" into the eye;
- Wait a few seconds after each drop without squeezing the bottle. This will enable enough air to flow back into the system, otherwise it is more difficult to get the drops out;
- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the proprieties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions. A small quantity of solution also always remains in the bottle;
- Shake off the remaining drop from the tip after use;

- After use close immediately the bottle with the cap;
- Do not put the protection cap back on when the container is still squeezed;
- Do not touch the applicator tip with anything, including your eye or your fingers;
- The product can be used up to 3 months after first opening;
- "BLUEMILLA" can also be used several times a day based on personal needs or a doctor's advice;
- "BLUEMILLA" can be used with contact lenses.

## WARNINGS AND PRECAUTIONS

- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if container is damaged or not perfectly closed when you open it for the first time;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- Do not use the product after the expiry date indicated on the package;
- Do not use in case of known incompatibility towards any of the ingredients and/or in case you are allergic to Asteraceae plants;
- If you suffer from severe damage to the clear layer at the front of the eye (the cornea), phosphates may cause in very rare cases cloudy patches on the cornea due to calcium build-up during treatment;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Dispose of the product according to your local waste management regulations;
- Keep out of the reach and sight of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- Do not try to tamper or disassemble the nozzle from the bottle;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

## SIDE EFFECTS

No undesirable side-effects are known so far.

## STORAGE CONDITIONS

Store below 30°C.

Content must be used within 3 months after the first opening.

## PACKAGING

One bottle of 10 ml of sterile solution preservative free.

## VALIDITY

See the expiry date reported on the box and on the container.



DT/01/DT/001



**C.O.C. Farmaceutici S.r.l.**

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Last Revision: 02.01 del 10/2022



**C.O.C. Farmaceutici srl**

Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**

Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476