

# HYALUMILLA ADVANCED

"HYALUMILLA ADVANCED" is a sterile, ophthalmic solution, with physiological pH, based pharmaceutical grade Hyaluronic Acid (as sodium hyaluronate), a polymer normally present in different areas of the human body.

Hyaluronic acid has viscoelastic, muco-mimetic and muco-adhesive properties that improve tear film stability and maintain ocular surface lubricated and hydrated.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

"HYALUMILLA ADVANCED" is compatible with soft and rigid contact lenses.

## INTENDED USE

"HYALUMILLA ADVANCED" moisturizes and lubricates eye relieving symptoms of irritation, dryness and foreign body sensation caused by:

- dry eye syndrome;
- mechanical factors (contact lenses use, after eye surgery);
- environmental factors (wind, smoke, pollen, dust, pollution);
- visual stress (excessive light, prolonged use of computers, pc and tablets).

## COMPOSITION

Sodium Hyaluronate 0,2%, Citric Acid Monohydrate, Trisodium Citrate Bihydrate, Sodium chloride, Water For Injection

## INSTRUCTION FOR USE

- Wash and dry your hands thoroughly before use;
- Open the protective aluminium pouch;
- Detach the single dose container from the strip and open the single-dose container by turning and then lifting off the closing cap.
- Tilt your head back and look up.
- Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye.

- Hold the single-dose container into other hand, turn the container upside down and apply slight pressure to the single-dose container to release 1 drop of "HYALUMILLA ADVANCED" into the pocket.
- If you think that you have missed the eye, then insert another drop.
- Repeat the process for the other eye, if required.
- After use immediately close the single-dose container with the cap: the product can be used up to 24 hours after first opening. In order to close it the cap must first be turned over.
- The product can be used as required or according to the advice of your doctor.

## **WARNINGS AND PRECAUTIONS**

- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if the aluminum pouch is damaged or if the single dose is not perfectly closed;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use the product after the expiry date indicated on the package;
- Do not use in case of known incompatibility towards any of the ingredients;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- Dispose of the product according to your local waste management regulations;
- Keep out of the reach and sight of children;
- The product is usually intended for continuous use for no more than 30 days; if symptoms persist consult your doctor;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- "HYALUMILLA ADVANCED" is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

## **SIDE EFFECTS**

Immediately after instillation may cause a slight temporary blurring of vision due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before any activity that requires clear vision.

## **STORAGE CONDITIONS**

Store between 2°C and 30°C.

The single dose content must be used within 24 hours after the first opening.

Store unused single doses in the aluminum pouch until needed.

## PACKAGING

From 5 to 30 re-closable single dose containing 0.5 ml of ocular solution.

## VALIDITY

See the expiry date reported on the box and on the the single-dose container.



DT/01/DT/006



**C.O.C. Farmaceutici S.r.l.**

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Last Revision: 02.01 del 10/2022



**C.O.C. Farmaceutici srl**

Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
P.IVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**

Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476