

OFTA GREEN

OFTA GREEN is a sterile, saline ophthalmic solution, with physiological pH.

The saline solution composed by sodium chloride and water for injection gives relief in case of dry and irritated eyes.

The formulation also contains Chamomile, Witch Hazel and Eye Bright distilled waters to ensure a moisturizing action.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

OFTA GREEN is compatible with contact lenses.

INTENDED USE

OFTA GREEN is used for its refreshing and soothing actions.

OFTA GREEN is indicated for relief of dry and irritated eyes due to:

- atmospheric agents and environmental factors (air conditioning, excessive solar radiation, wind, smoke, pollution, salt, sea water, pool water);
- excessive fatigue due to long hours working on computers, studying;
- mild dry eye syndrome;
- use for many hours a day of contact lenses.

COMPOSITION

Sodium Chloride (0.85%), Chamomile Distilled Water, Witch Hazel Distilled Water, Eye Bright Distilled Water, Sodium Phosphate Monobasic Monohydrate, Sodium Phosphate Dibasic Dihydrate, Water For Injection.

INSTRUCTIONS FOR USE

- Wash and dry your hands thoroughly before use;
- Tear off the tamper-evident seal before the first administration;
- Remove the protective cap and open the bottle;
- Turn the bottle upside down and exert slight pressure on the bottle to instill 1 or 2 drops in each eye;
- Wait a few seconds after each drop without squeezing the bottle. This will enable enough air to flow back into the sterile air filter system, otherwise it is more difficult to get the drops out;
- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system that act to preserve the sterility and the chemical and physical characteristics of preservative free solutions. A small quantity of solution also always remains in the bottle;

- Shake off the remaining drop from the tip after use;
- After use close immediately the bottle with the cap;
- Do not put the protection cap back on when the container is still squeezed;
- Do not touch the applicator tip with anything, including your eye or your fingers;
- The content should be re-used within the next 3 months from the first opening;
- It is suitable for frequent use. Unless your doctor has told you differently, you can use the product even several times a day (3 or 4 times or more often if required).

WARNINGS AND PRECAUTIONS

- If you suffer from severe damage to the clear layer at the front of the eye (the cornea), phosphates may cause in very rare cases cloudy patches on the cornea due to calcium build-up during treatment;
- The product is for ophthalmic use only;
- The product is for single patient use;
- Do not use the product if the container results, at the time of first opening, damaged or not perfectly sealed;
- Do not use if are visible particles and/or changes in the colour of the solution;
- During application, do not touch the eye or any other surface with the container's dispensing edge;
- Do not use the product after the expiry date indicated on the package and on the label;
- In case of side effects during treatment interrupt the use and consult a doctor;
- Do not use the product in case of hypersensitivity towards any of the components of the solution and/or Asteraceae plants and/or Compositae plants and/or Hamamelidaceae plants and/or Orobanchaceae plants;
- Dispose of the product according to your local waste management regulations;
- Keep out of reach of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- Do not try to tamper or disassemble the nozzle from the bottle;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

SIDE EFFECTS

No undesirable side-effects are known so far.

STORAGE CONDITIONS

Store between 2°C and 25°C in a dry place away from heat and direct exposure to sunlight. The content must be used within 3 months after the first opening.

PACKAGING

One bottle of 10 ml of sterile solution preservative free.

VALIDITY

See the expiry date reported on the box and on the container.



DT/01/DT/011



C.O.C. Farmaceutici S.r.l.

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Distributor's address:

Last Revision: 03.01 del 05/2023



C.O.C. Farmaceutici srl
Via Modena, 15
40019 - S. Agata Bolognese (BO) ITALIA
Via Chiesa Sud, 156 C/D/E/F/G
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.
C.F. e Iscr. Reg.
Impr. di Bologna 03541870378
P.IVA IT 01603491208
R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.
Via Costarica, 20/22
00071 Pomezia (Roma)
ITALIA

cap. soc. € 472.500,00 I.V.
C.F. / P.IVA 05406661008
R.E.A. Roma N. 883476