

# OFTA GREEN

OFTA GREEN is a sterile, saline ophthalmic solution, with physiological pH.

The saline solution composed of sodium chloride and water for injection gives relief in case of dry and irritated eyes.

The formulation also contains Chamomile, Witch Hazel and Eye Bright distilled waters to ensure a moisturizing action.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

OFTA GREEN is compatible with contact lenses.

## INTENDED USE

OFTA GREEN is used for its refreshing and soothing actions.

OFTA GREEN is indicated for relief of dry and irritated eyes due to:

- atmospheric agents and environmental factors (air conditioning, excessive solar radiation, wind, smoke, pollution, salt, sea water, pool water);
- excessive fatigue due to long hours working on computers, studying;
- mild dry eye syndrome;
- use for many hours a day of contact lenses.

## COMPOSITION

Sodium Chloride (0.85%), Chamomile Distilled Water, Witch Hazel Distilled Water, Eye Bright Distilled Water, Sodium Phosphate Monobasic Monohydrate, Sodium Phosphate Dibasic Dihydrate, Water For Injection.

## INSTRUCTIONS FOR USE

- Wash and dry your hands thoroughly before use;
- Open the protective aluminum pouch;
- Detach the single-dose container from the strip;
- Twist and remove the cap to break the seal;
- Place the cap taking care to put it on a clean surface;
- Instill 1/2 drops in each eye by gently squeezing the single dose;
- After use immediately close the single-dose container: the content must be used within 12 hours after the first opening;
- It is suitable for frequent use. Unless your doctor has told you differently, you can use the product even several times a day (3 or 4 times or more often if required).

## WARNINGS AND PRECAUTIONS

- If you suffer from severe damage to the clear layer at the front of the eye (the cornea), phosphates may cause in very rare cases cloudy patches on the cornea due to calcium build-up during treatment;
- The product is for ophthalmic use only;
- The product is for single patient use;
- Do not use the product if container is damaged or not perfectly closed when you open it for the first time;
- Do not use if are visible particles and/or changes in the colour of the solution;
- During application, do not touch the eye or any other surface with the dropper of the container;
- Do not use the product after the expiry date indicated on the package;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use in case of known incompatibility towards any of the components of the solution and/or Asteraceae plants and/or Compositae plants and/or Hamamelidaceae plants and/or Orobanchaceae plants;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- Dispose of the product according to your local waste management regulations;
- Keep out of the reach of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

## SIDE EFFECTS

No undesirable side-effects are known so far.

## STORAGE CONDITIONS

Store between 2°C and 25°C in a dry place away from heat and direct exposure to sunlight. The content must be used within 12 hours after the first opening.

## PACKAGING

From 5 to 30 re-closable single-dose of 0.5 ml

## VALIDITY

With un-open pack, not exceed the expiry date stated on the box and on the single dose.



DT/01/DT/011



**C.O.C. Farmaceutici S.r.l.**

Via Modena, 15 - 40019

Sant'Agata Bolognese (BO) Italia

Distributor's address:

---

---

Last Revision: 03.01 del 05/2023



**C.O.C. Farmaceutici srl**

Via Modena, 15  
40019 - S. Agata Bolognese (BO) ITALIA  
Via Chiesa Sud, 156 C/D/E/F/G  
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1200.000 I.V.  
C.F. e Iscr. Reg.  
Impr. di Bologna 03541870378  
PIVA IT 01603491208  
R.E.A. Bologna N. 299432



**Tubilux Pharma S.p.A.**

Via Costarica, 20/22  
00071 Pomezia (Roma)  
ITALIA

cap. soc. € 472.500,00 I.V.  
C.F. / P.IVA 05406661008  
R.E.A. Roma N. 883476