

CARBOCARE GEL

Lubricating eye gel
10 gr tube

Carbocare gel is a sterile solution containing Carbopol 980 at 0.2%, capable of lubricating the ocular surface, forming a light protective film that is slowly removed by blinking. The product has no immunological-pharmacological activity but only mechanical or physical action.

INDICATIONS

Carbocare gel brings prompt and lasting relief in case of sensation of dryness, irritated and tired eyes caused by external factors such as prolonged exposure to video screens, air conditioning, dust, pollution, smoke, etc in wearers and non-wearers of contact lenses.

COMPOSITION

Carbopol 980 0.2%, Cetrimide 0.01%, Disodium edetate, Sorbitol, Sodium hydroxide, Purified water.

INSTRUCTIONS FOR USE

Wash and rinse your hands thoroughly. Hold the tube upside down and press lightly until a small drop forms. Pull the lower eyelid down slightly to form a pocket between your eyelid and eye and place the drop in your eye. Blink two or three times so that the solution spreads and forms a resistant transparent film on the eye surface. Apply one or two drops in the eye, three or four times a day as needed or follow the instructions given by your practitioner.

WARNINGS AND PRECAUTIONS

- Do not use it if you are hypersensitive (allergic) to any of the constituents.
- To avoid possible blurring it is recommended to remove contact lenses before application.
- Do not instill Carbocare gel in the eye(s) together with other substances as their effect may be altered.
- To prevent possible contamination, do not let the tip come into contact with any surface.
- Do not use it if the tube or security seal is damaged.
- Do not share the product to prevent cross contamination.
- Stop using the product if you experience pain, redness or any change in vision following the instillation. Ask a doctor for advice if the symptoms persist or get worse.

- If you experience blurred vision after application, wait until your vision is clear before driving or using tools or machinery.
- Keep out of reach of children.
- Do not use the product after the expiry date.

PACKAGE

10g tube.

STORAGE CONDITIONS

- Store the tube at a temperature below 30°C.
- Do not use the product for more than 30 days after the tube has been opened.
- Any residual product must be disposed of.



Tubilux Pharma S.p.A Via Costarica 20/22 00071 Pomezia (RM) Italy
MEDICAL DEVICE

DISTRIBUTED BY:

Last Revision Rev.03 of 09/2016



C.O.C. Farmaceutici srl
 Via Modena, 15
 40019 - S. Agata Bolognese (BO) ITALIA
 Via Chiesa Sud, 156 C/D/E/F/G
 41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.
 C.F. e Iscr. Reg.
 Impr. di Bologna 03541870378
 P.IVA IT 01603491208
 R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.
 Via Costarica, 20/22
 00071 Pomezia (Roma)
 ITALIA

cap. soc. € 472.500,00 I.V.
 C.F. / P.IVA 05406661008
 R.E.A. Roma N. 883476