

MIRTHILLA BASE

Lubricating ophthalmic solution with sodium hyaluronate.
10 ml into a multidose ophthalmic container.

Mirthilla base is a preserved sterile hypotonic eye solution, containing sodium hyaluronate at 0.2%.

Mirthilla base maintains and restores the physiological conditions of the eye surface, protects its cells, induces a long-lasting comfort.

INDICATIONS

Treatment of the dry eye symptoms.

INTENDED PURPOSE

Mirthilla base brings prompt and lasting relief in case of sensation of dryness, irritated and tired eyes caused by dry eye disease and by external factors such as prolonged exposure to video screens, air conditioning, dust, pollution, smoke, etc.

Mirthilla base can be used by wearers of all types of corrective contact lenses.

COMPOSITION

Sodium Hyaluronate 0,2 %, Sodium chloride, Boric Acid, Borax, Dequest, Sodium Perborate (0.03%) and Purified Water.

INSTRUCTIONS FOR USE

- Wash and rinse your hands thoroughly.
- Remove the protective cap. Make sure that the tip of the bottle does not touch your fingers.
- Lean your head back slightly and pull your lower eyelid down slightly with a finger of your free hand and hold the bottle vertically above your eye with the nozzle pointing downwards. Then gently squeeze the bottle to apply one or two drops into each eye, or according to the specialist's advice. Avoid contact of the top of the bottle with your eye or your fingers to prevent any possible contamination of the solution.
- Replace the protective cap after use.
- The product can be used whenever necessary.

WARNINGS AND PRECAUTIONS

- Do not use it if the bottle is damaged.
- Do not share the bottle to prevent cross contamination.
- If you have to apply additional treatments to the eye, an application interval of at least 15 minutes should be left between the two medications.
- To prevent possible contamination, do not let the tip of the dropper come into contact with any surface.
- Do not use it if you are hypersensitive (allergic) to any of the constituents.
- Rare cases of mild eye irritation are possible after product instillation. Stop using it if you experience pain, redness or any change in vision following the installation. Ask a doctor for advice if the symptoms persist or get worse.
- If you experience blurred vision after application, wait until your vision is clear before driving or using tools or machinery.
- Do not swallow the solution.
- Keep out of reach of children.
- If you experience a persistent problem, please consult your doctor/optician.
- Do not instill Mirthilla base in the eye(s) together with other substances as their effect may be altered.

VALIDITY

Do not use the solution after the expiry date.

PACKAGE

10 ml into a multidose ophthalmic container.

STORAGE CONDITIONS

- Store the bottle at a temperature below 30°C.
- After the bottle has been opened, the solution must be used within 90 days. Any residual product must be disposed of.



STERILE	A
---------	---

Tubilux Pharma S.p.A

Via Costarica 20/22 00071 Pomezia (RM) Italy

MEDICAL DEVICE

DISTRIBUTED BY:

.....

.....

Last Revision Rev.01 of 05/2022



C.O.C. Farmaceutici srl

Via Modena, 15
40019 - S. Agata Bolognese (BO) ITALIA
Via Chiesa Sud, 156 C/D/E/F/G
41016 - Rovereto s/S - Novi di Modena (MO) ITALIA

Cap. Soc. € 1.200.000 I.V.
C.F. e Iscr. Reg.
Impr. di Bologna 03541870378
P.IVA IT 01603491208
R.E.A. Bologna N. 299432



Tubilux Pharma S.p.A.

Via Costarica, 20/22
00071 Pomezia (Roma)
ITALIA

cap. soc. € 472.500,00 I.V.
C.F. / P.IVA 05406661008
R.E.A. Roma N. 883476